

Product: Cobra Knee Brace

FEATURES:

- Designed for patients with mild to moderate osteoarthritis of the knee, the Cobra is a low-profile brace with 3 points of leverage for reducing pressure inside the knee joint.
- Universal size.
- Extremely light weight.
- The strapping system allows for full adjustment, with quick-release snaps for easy application and removal.
- Valgus and Varus adjustment to fit the patient's unique alignment.



Order #	Location
900-L	Left Medial
900-R	Right Medial
901-L	Left Lateral
901-R	Right Lateral

HOW TO APPLY: Open all straps and step through the brace. With leg slightly bent, center hinge one (1) inch above the top of the knee cap. Push hinge slightly back behind the midline of the knee. Attach the first strap label on the number system shown on the straps. Pull the strap on top of the knee area, making sure it is snug. Attach all the straps on the buckle, making sure it is snug. Stand up and walk, making sure it is not too tight or too loose.

CARE INSTRUCTIONS: Hand wash in cold water with mild soap. Air dry away from heat. Don't tumble dry. Don't bleach.

CAUTION: Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGGESTED HCPCS CODE: L1843