



Application Instructions

PRODUCT: Active Ankle Sleeve

SKU: 305X

- Fits Left or Right Ankle
- Support with Compression
- Reduce Swelling & Pain



Order #	Size	Ankle Circumference
3053	Small	9 ¹ / ₂ " - 11"
3055	Medium	11" - 12"
3057	Large	12" - 13"
3058	X-Large	13" - 14"
3059	2X-Large	14" - 15"

How to Apply

1. Pull the support up over the foot with the heel snugly towards the back.
2. The support should fit snug but not so tight to restrict circulation.

CARE INSTRUCTIONS:

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air-dry away from heat. Do not tumble dry. Do not bleach.

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling

Advanced Orthopaedics, Inc.
326 Main Street Harleysville, PA 19438
800.270.7074
info@advanced-orthopaedics.com

This QR code can be scanned by your smart phone and directs you to the Product Detail Page.

