



# Application Instructions

**PRODUCT:** Active Elbow Sleeve

**SKU:** 304X

- Sizes Small - X Large
- Fits Left or Right Elbow
- Support with Flexibility
- Compression Support Sleeve for Tendonitis & Tennis Elbow Pain
- Reduce joint pain during any activity
- Provides superior support, comfort and relief



Order #	Size	Elbow Circumference
3043	Small	9" - 11"
3045	Medium	11" - 12 <sup>1</sup> / <sub>2</sub> "
3047	Large	12 <sup>1</sup> / <sub>2</sub> " - 14"
3048	X-Large	14" - 16"

## How to Apply

1. Pull the support up over the arm and center on the elbow joint.
2. The support should fit snug but not so tight to restrict circulation.

## CARE INSTRUCTIONS:

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air-dry away from heat. Do not tumble dry. Do not bleach.

## CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling

Advanced Orthopaedics, Inc.  
326 Main Street Harleysville, PA 19438  
800.270.7074  
info@advanced-orthopaedics.com

This QR code can be scanned by your smart phone and directs you to the Product Detail Page.

