



Application Instructions

PRODUCT: Active Wrist Brace

SKU: 3030

- Universal Size
- Fits Left or Right Wrist
- Compression Support
- Helps relieve your wrist pain and provide fast recovery
- Targeted compression to the wrist area
- Support for Carpal Tunnel Syndrome, RSI, wrist pain & strain
- Lightweight and breathable



Order #	Size	Wrist Circumference
3030	Universal	5" - 10"

How to Apply

1. Wrap around the wrist until snug.
2. The support should fit snug but not too tight as to be uncomfortable or restrict circulation.
3. Attach the Velcro closure to secure in place.

CARE INSTRUCTIONS:

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air-dry away from heat. Do not tumble dry. Do not bleach.

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling

Advanced Orthopaedics, Inc.
326 Main Street Harleysville, PA 19438
800.270.7074
info@advanced-orthopaedics.com

This QR code can be scanned by your smart phone and directs you to the Product Detail Page.

