



Application Instructions

Product: Pediatric Walker

Model: 3X0-P

- Excellent for infants
- Extremely lightweight
- Soft, plush lining
- Fits left or right foot



Order #	Size	Years of Age	Sole Length
330 - P	Small	1 - 2	Up to 6"
360 - P	Medium	2 ¹ / ₂ - 6	6 ¹ / ₂ " - 7 ¹ / ₂ "
390 - P	Large	6 ¹ / ₂ - 9 ¹ / ₂	7 ¹ / ₂ " - 8 ¹ / ₂ "

SIZING: Measure according to sole length. Fits either left or right foot.

HOW TO APPLY: Remove the liner from the brace. Wrap the liner around the patient's leg and foot. Ensure the heel fits snugly into the portion first followed by the calf portion. The wrap should fit snug, but not too tight. Note - it may be necessary to adjust the fit of the wrap if swelling or edema decreases during the healing process. Attach one end of the straps onto outside of the struts. In a seated position spread the side uprights of the brace and have the patient step into the boot. The foot should be positioned flat with the heel in back of the boot. Affix the uprights to the liner. The uprights should be positioned parallel to the leg. Close the straps starting at the toes and working up the leg. Adjust the straps if necessary.

CARE INSTRUCTIONS: Remove the liner from the splint. Hand wash in warm water with mild soap. Air dry away from heat. Do not machine wash or dry. Do not bleach. Spot clean the splint and straps with a damp cloth if needed.

CAUTION: Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGG HCPCS CODE: L4386/L4387

