



**Product: Air Tennis Elbow System**  
**Model: 2301**

- Air tennis elbow system has three uniquely designed features
- Air therapy pad helps reduce swelling
- Magnetic therapy pad improves circulation
- Foam tension pad to maximize compression and dissipate force
- Fits either left or right forearm



<b>Order#</b>	<b>Size</b>
2301	Universal

## HOW TO APPLY:

Open the strap and slide over the forearm so it is positioned just below the elbow with the air-gel pad centered on the tendon where compression is needed. Close the tension strap so it fits comfortably without restricting circulation. Adjust as needed.

## CARE INSTRUCTIONS:

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water {do not twist or wring}. Air dry away from heat. Do not tumble dry. Do not bleach.

## CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

