



Application Instructions

PRODUCT: Recovery Shoulder System

SKU: 2700

- Universal size, left or right
- Quick release buckle
- Breathable mesh material
- Extra padding around the neck for increase comfort
- Thumb rest to reduce migration



Order #	Size	Forearm Length
2700	Universal	11.5 - 17.5"

How to Apply

1. May need assistance when applying and removing.
2. Unbuckle the shoulder strap from and release the Velcro strap to open up the pouch sling.
3. Place the injured arm in the pouch sling.
4. Bring strap across back and over the opposite shoulder and attach the buckle.
5. The shoulder strap length can be adjusted by removing the Velcro at each end of the strap and placing back into the appropriate position.
6. Replace the Velcro strap to close the pouch sling.
7. To adjust the length of the pouch sling, remove the additional Universal pouch and replace the velcro in the appropriate position.
8. If needed, place your thumb in the thumb loop.

CARE INSTRUCTIONS:

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air-dry away from heat. Do not tumble dry. Do not bleach.

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGG HCPCS CODE: L3670