



Application Instructions

PRODUCT: Recovery Shoulder Pillow w/Ball
SKU: 2770

- Universal size, left or right
- Quick release buckle
- Pillow w/Ball Included
- Extra padding around the neck for increase comfort
- Thumb rest to reduce migration



Order # 2770 Size Forearm Length

Universal 11.5 - 17.5"

How to Apply

1. May need assistance when applying and removing.
2. Unbuckle the shoulder strap from and release the Velcro strap to open up the pouch sling.
3. Place the injured arm in the pouch sling.
4. Bring strap across back and over the opposite shoulder and attach the buckle.
5. The shoulder strap length can be adjusted by removing the Velcro at each end of the strap and placing back into the appropriate position.
6. Replace the Velcro strap to close the pouch sling.
7. To adjust the length of the pouch sling, remove the additional Universal pouch and replace the velcro in the appropriate position.
8. If a neutral pillow or 15° abduction pillow is included, place the pillow at the waistline. Attach the sling to the pillow, lining up hook material on the pillow with the loop on sling. Adjust the waist strap to fit.
9. Place the double sided hook material on the inside of the cut strap end and secure it to the waist strap.
10. If needed, place your thumb in the thumb loop.

CARE INSTRUCTIONS:

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air-dry away from heat. Do not tumble dry. Do not bleach.

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGG. HCPCS CODE: L3670

Advanced Orthopaedics, Inc.
326 Main Street Harleysville, PA 19438
800.270.7074
info@advanced-orthopaedics.com

This QR code can be scanned by your smart phone and directs you to the Product Detail Page.

