



Product: Thumb Splint Brace

Model: 21003

- Perforated suede thumb splint provides thumb immobilization and protection.
- Constructed of lightweight, breathable, padded material for maximum comfort and wear.
- Perforated to allow moisture to escape the skin area.
- Malleable, rigid thumb stay is adjustable for a custom fit.
- Hook and loop closure for easy adjustment.
- Fits either left or right thumb.



Order#	Size
21003	Universal

SIZING:

Universal design fits a range of sizes. Fits right or left thumb.

HOW TO APPLY:

Unfasten closures. Align the splint so the metal stay is parallel to the outside of the thumb (medial/inner side of the wrist). The metal stay can be bent to shape for a customized fit. The spica portion should fit around the base of the thumb. Using free hand, close the top strap around the thumb and wrap the straps around the wrist and hand starting at the bottom. Readjust if necessary.

CARE INSTRUCTIONS:

Fasten closures. Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGG HCPCS CODE: L3999

AdVanced Orthopaedics, Inc.
326 Main Street Harleysville, PA 19438
800.270.7074
info@advanced-orthopaedics.com

This QR code can be scanned by your smart phone and directs you to the Product Detail Page.

