

Application Instructions

Product: Elastic Slip-On Elbow Support Model: 231X

- Easy slip-on.
- Seamless construction will not irritate skin.
- Durable knitted elastic provides support with flexibility.
- Provides intermediate compression to the elbow area.



Order #	Size	Forearm Circumference
2313	Small	8.5" - 10"
2315	Medium	10" - 11.5"
2317	Large	11.5" - 13"
2318	X-Large	13" - 14.5"

SIZING: Measure around the widest part of the forearm.

HOW TO APPLY:

Pull the support up over the arm and center on the elbow joint. The support should fit snug but not so tight to restrict circulation.

CARE INSTRUCTIONS:

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGG HCPCS CODE: A4466

Advanced Orthopaedics, Inc. 326 Main Street Harleysville, PA 19438 800.270.7074 info@advanced-orthopaedics.com