



## **Product: Universal Rib Belt**

**Models: 2400, 2410**

- Constructed of 6" wide breathable elastic material.
- Helps stabilize rib and sternum fractures by limiting chest expansion through compression.
- Female belt has contours around bust line and male belt is straight.



<b>Order#</b>	<b>Size</b>	<b>Rib Measurement</b>
2400	Universal Male	24" - 60"
2410	Universal Female	24" - 60"

**SIZING:** Measure around your center, just below bust.

### **HOW TO APPLY:**

Apply in a standing or laying position. Open belt to its fullest length with the care tag on the inside. Position the rib belt around the rib cage. Stretch both sides and wrap the sides around the body. Secure the support where needed for desired compression. Support should fit snug but not too tight. Readjust if necessary.

### **CARE INSTRUCTIONS:**

Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

### **CAUTION:**

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

**Advanced Orthopaedics, Inc.**

326 Main Street Harleysville, PA 19438

800.270.7074 [info@advanced-orthopaedics.com](mailto:info@advanced-orthopaedics.com)

This QR code can be scanned by your smart phone and directs you to the Product Detail Page.

