



Product: Deluxe Airprene Knee Brace

Model: 80X

- Perforated “neoprene-like” material allows moisture to escape away from the skin.
- Two medial and two lateral spiral steel stays for stability with flexion.
- Universal buttress pad for easy adjustment and exact fit.
- Tapered in the back for a comfortable fit.
- Piping along thigh and calf areas of support to help stop flexing.



Order #	Size	Knee Circumference
803	Small	12½" - 14½"
805	Medium	14" - 16"
807	Large	15½" - 18"
808	X-Large	17½" - 20"
809	2X-Large	19½" - 22"
810	3X-Large	21" - 24"

SIZING: Measure around the center of the knee with leg extended.

HOW TO APPLY:

Unfasten all straps and closures. Pull the brace over the knee with the opening centered on the kneecap and with the medial/lateral flexible stays aligned with the sides of the knee joint. Secure straps beginning with the top strap. Feed straps through rings and secure the hook and loop (Velcro-like) closure. Ensure support is not too tight.

CARE INSTRUCTIONS: Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

CAUTION: Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGG HCPCS CODE: A4466

Advanced Orthopaedics, Inc.
326 Main Street Harleysville, PA 19438
800.270.7074 info@advanced-orthopaedics.com