



# Application Instructions

Product: The Falcon™ Ankle Brace  
Model: 82X-R / 83X-L

## Size Chart

Order # Left	Order # Right	Size	Male Shoe Size	Female Shoe Size
833 - L	823 - R	Small	Up to 8	Up to 9
835 - L	825 - R	Medium	8½" - 12"	9½" - 13"
837 - L	827 - R	Large	12+	13+

Sizing: Use your proper shoe size

### HOW TO APPLY:

Unfasten Top Closure cuff and soft liner. WHILE SEATED, place the injured foot in the brace, similar to putting on a shoe. Position so that the heel is against the back of the soft liner.

Fasten the Velcro closure on the soft liner. Position so that the rigid plastic stays on the brace are centered over each side of the ankle.

Fasten the cuff closure straps over the soft liner so that the brace fits snug and little movement is experienced.

Simultaneously pull both quick-lace straps tight and secure by crossing each quick-lace strap over the top of the foot and attaching to the soft liner. Make sure the brace still fits snug and is centered over each side of the ankle.

### POSTERIOR PANEL APPLICATION (OPTIONAL)

or added support, the optional posterior panel can be attached to the Accord III. Open the plastic cuff at the tabs provided (when facing the back of the brace, tabs will be located on the right, just behind the Velcro tabs for the cuff closure). and re-attach the cuff. Apply the brace as outlined in the section above.

### CARE INSTRUCTIONS:

Remove the soft liner from within the plastic shell. Hand wash in cold water with mild detergent. AIR DRY ONLY. Wipe the remainder of the brace with cold water. Allow all parts to air dry before re-assembly. Do not dry clean, iron, or bleach any part of this brace, as it could cause damage and void the manufacturer's warranty.

### CAUTION:

Before using this device, please consult your physician. Be sure to thoroughly read and understand the directions as laid out above. Correct application is vital to the proper function of this device.

