



Application Instructions

Product: ACL Knee Brace


Model: 84X-R, 85X-L

- Strong, lightweight composite frame
User friendly ROM hinge for quick and easy adjustment
- Malleable uprights and drop-lock ROM Hinge



Order # Right	Order # Left	Size	Thigh Circumference
843-R	853-L	Small	15 ¹ / ₂ " - 18 ¹ / ₂ "
845-R	855-L	Medium	18 ¹ / ₂ " - 21"
847-R	857-L	Large	21" - 23 ¹ / ₂ "
848-R	858-L	X Large	23 ¹ / ₂ " - 26 ¹ / ₂ "
849-R	859-L	2X Large	26 ¹ / ₂ " - 29"

APPLICATION INSTRUCTIONS:

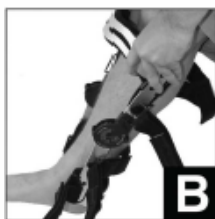

1. Lay the brace on a flat surface and loosen all of the closure straps (Fig. A)
2. Place the brace over the leg as shown. Make sure the range-of-motion (ROM) hinges are centered on each side of the knee (Fig. B)
3. Tighten and secure each support strap, starting from the top of the brace. Make sure that the ROM hinges continue to stay aligned with the knee (Fig. C). Tighten and re-adjust as needed.
4. Adjust the ROM hinge by sliding the drop-lock to the desired degree and releasing to set in place. Do this for both the flexion and extension adjustments (Fig. D).

CARE INSTRUCTIONS: Hand wash in cold water with mild soap. Wrap in a towel to remove excess water. Air dry away from heat.

CAUTION: Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGG HCPCS CODE: L1845 and L1852