

Product: Coral Soft OA Knee Brace

FEATURES:

- Adjustable Velcro hook and lock straps
- Low-profile, light-weight brace provides 3 points of leverage to reduce pressure inside the knee joint
- Ideal for unilateral compartment OA knee users
- Valgus and Varus adjustment to fit individual's alignment

Order

950-L

950-R


Location

Fits Left Medial, Right Lateral


Fits Right Medial, Left Lateral




HOW TO APPLY:

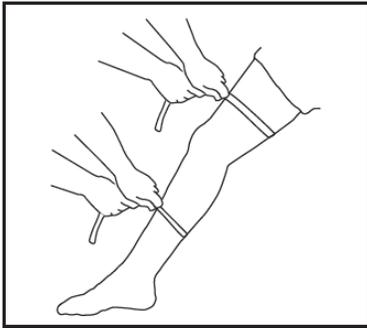


1. Measure circumference of thigh and calf with leg extended.

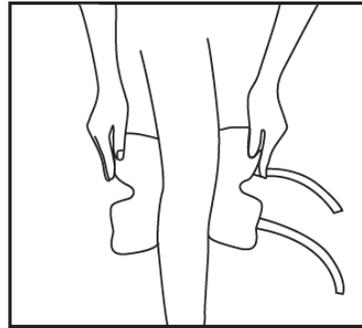

2. Unfasten all straps and open the brace.

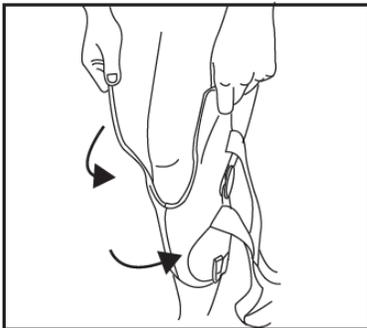

3. Wrap sleeve around knee and secure with hook and loop closure.
 4. Position the circular cutting of sleeve onto the knee cap.
 5. Make sure the axis of the hinges line up with the center of your knee joint as you bend your knee.

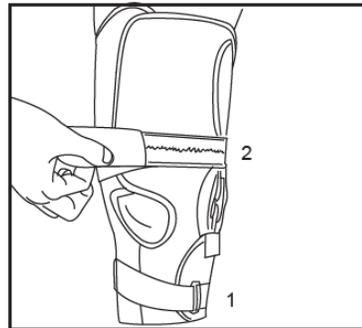

6. Pass the circumferential strap 1&2 through buckles and secure snugly.

CARE INSTRUCTIONS: The rigid components can be wiped clean with a damp cloth. Soft goods can be hand washed in warm water with a mild detergent. Triple rinse and air dry. Avoid twisting the material to remove water.

Note : Your practitioner is familiar with your specific medical condition and is using this product as part of your total care. Make sure to follow their directions exactly, even if they do not match the directions below.

CAUTION: The application instructions listed above are intended for the use of qualified professionals only. It is not recommended for patients attempt to apply or adjust this device unless otherwise instructed to do so by an orthopedic professional. Any questions regarding further adjustment or frequency/ duration of wear should be directed to your orthopedic professional, as they know the individual condition for which this device was prescribed.

SUGGESTED HCPCS CODE: L0626